senza Conservanti / Preservative free / bez Konserwantów

rely[®] 101

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Rev. 00 - 22/11/2024

en

RELYS® is a medical device that can be used as an adjunct to therapy in the multifactorial Dry Eye Disease (DED), defined by international experts of the research and education organisation, Tear Film and Ocular Surface Society (TFOS), as:

"A multifactorial disease of the ocular surface characterised by a loss of tear film homeostasis and accompanied by ocular symptoms, in which tear film instability and hyperosmolarity, ocular surface inflammation and lesions, and sensorineural abnormalities play etiological roles" (Ocular Surface Journal, 2017).

RELYS® is an ophthalmic solution for topical use containing **sT-LysYal®**, a Supramolecular Aggregate capable of acting on the Ocular Surface structures by restoring its Homeostasis through a significant increase in the hydration processes, which modulate inflammation and allow for repairing the lesions of the corneal and conjunctival epithelia.

Unlike Sodium Hyaluronate, which due to the "Chain Folding" effect, reduce water mobility, the **sT-LysYal®** Supramolecular Aggregate is characterised by a remarkable mobility of water along its chains with the relative exchange of the latter with the surrounding intra and extra cellular compartments (HYDRATION).

INDICATIONS

In all situations of DED (Dry Eye Disease) and alterations of the Ocular Surface, **sT-LysYal®**, thanks to its hydrodynamics, restores correct HOMEOSTASIS by supporting the restoration of the physiological tear film, even in the presence of contact lenses.

RELYS® is indicated in cases of:

- Dry Eye Disease (DED moderate and severe)
- Dry Eye Disease in patients with diabetes
- DED secondary to surgical procedures (cataract, refractive, glaucoma, vitrectomies)
- corneal epithelial lesions
- blepharitis
- eye redness
- excessive exposure to the sun
- visual fatigue
- prolonged use of VDUs (blue light)
- contact lens wearers
- patients with an artificial lens
- burning, itching and gritty sensation in the eyes
- excessive tearing
- exposure to irritative phenomena (excessive exposure to dust, smoke, wind, driving, air pollution, etc.)

By correctly hydrating all the structures of the Ocular Surface and maintaining their homeostasis, **RELYS®** stabilizes the physiological conditions useful for preventing and counteracting the onset of DED in patients subject to predisposition and adverse environmental conditions. By keeping the eyes well hydrated, **RELYS®** consequently counteracts the irritating effects and eyestrain, thus immediately providing a cooling effect and improving the quality of vision.

WARNINGS

Product for ophthalmic use only. Wait 10-15 minutes after use before instilling other products for ophthalmic use.

When instilled, it may be experienced a temporary blurred vision that disappears after few seconds. In this event, wait for the blurring to disappear before undertaking any activity that requires visual clarity (driving cars, using machines, etc.).

Do not dispose of the container in the environment after use. Do not use if container appears already opened or damaged when first opened. Do not use after the expiry date indicated on the package. Do not swallow. Keep out of the reach and sight of children.

CONTRAINDICATIONS AND SIDE EFFECTS

Currently, there are no known contraindications for the use of the device. However, it is recommended to avoid using in case of hypersensitivity to one of the components. There are no known clinically relevant side effects. A burning sensation may be felt as soon as it is applied, which tends to resolve spontaneously within a short time. In the event of side effects other than those described, suspend application and consult your doctor.

Any serious incident that occurs in relation to the device must be reported to the Manufacturer, and to the competent authority of the user's and/or patient's Member State.

PREGNANCY AND BREASTFEEDING

Although there are no known effects of **RELYS®** during pregnancy and/or breastfeeding, the application of the product in these cases is subject to your doctor's opinion.

COMPOSITION

sT-LysYal® 0,32%, mannitol, sodium citrate, citric acid, aqua.

STORAGE METHODS

Keep away from heat sources. It is recommended to store at room temperature.

PACKAGES

Box containing 20 mini-containers of 0.35 ml

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Separare
Tear apart
Oddzielić

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en

INSTRUCTIONS FOR USE of the SINGLE-DOSE

Open the protective pouch and detach a mini-container.

Open the mini-container carefully by twisting and pulling the cap and avoiding contact with the opening.

Gently press the mini-container and instil 1–2 drops per eye, being careful not to touch the eye with the tip of the mini-container.

Close the mini-container carefully and store it inside the pouches.

The remaining liquid may be used within 12 hours after first opening.

It is possible to repeat the operation several times a day as needed.

It is advisable to continue using **RELYS®** for a few days after symptoms have subsided.

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